ATHENA CHAMPION®

Replacement Turbine for Midwest Quiet-Air Lever

Item Number: 70903

INSTALLATION INSTRUCTIONS

Read these instructions before proceeding to replace the turbine.

Before replacing the turbine, carefully examine handpiece head for dents. If there are any dents in the head, return the entire handpiece to your dealer for repairs. **DO NOT INSTALL** replacement turbine into dented handpiece. Installation will permanently damage turbine.

1. Remove cap by turning counterclockwise, using the cap wrench.
2. To remove worn turbine, press the blunt end of an inserted shipping post or bur against a hard surface.
3. Remove all foreign material and debris from interior of handpiece head.
4. Remove original end cap from old turbine. If the quad ring (O-ring) remains, carefully remove. **CLEAN & LUBRICATE** interior of head and cap.
5. Insert new quad ring (O-ring) from packaging. By pinching the quad ring between the fingers, insert into clean end cap.
6. Insert new turbine assembly making sure the quad ring is positioned properly on the flanged bearing (the bearing closest to the chuck pin).
7. Begin to thread the end cap into the head of the handpiece until almost flush to the last thread. To properly seat the turbine, gently pull and wiggle the chuck pin into the bottom of the head.
8. Tighten the end cap down with a cap wrench taking care to prevent cross threading.

9. Turbine should spin freely. If not, remove turbine and reinstall according to steps 5, 6 & 7. If the turbine does not spin freely, **DO NOT** run handpiece. Make sure the turbine is seated flush and straight, and that the cap is properly tightened.
10. Install bur securely into handpiece. **DO NOT OPERATE WITHOUT BUR!** Lubricate handpiece prior to use.

STERILIZATION PROCEDURES

Lubricate the handpiece prior to sterilization. Place 2 to 3 drops of lubricant into the drive air (air inlet) tube. Run the handpiece for 6 to 10 seconds as you would normally. Wipe any excess lubricant and autoclave as normal. Prior to autoclaving, we recommend removing the bur and any attachments from the handpiece. Insert the handpiece into a sterilization pouch. Place into a steam autoclave. Sterilize at 135°C (275°F) per autoclave manufacturer's parameters. *Do not let the handpiece remain in pouch under wet or damp conditions. Such conditions can result in corrosion of internal parts and shorten the life of the handpiece. Do not use cold sterilization solutions containing acids or chlorine. Do not immerse the handpiece in water or disinfection solutions. Do not place the handpiece in an ultrasonic bath.*

LUBRICATE BEFORE USE

After sterilization and cool down time, insert bur and tighten with bur tool. Lubricate again and briefly operate to expel excess lubricant. Wipe dry any excess.

Warranty: This part is warranted to the original purchaser against defective workmanship and materials under normal installation, use and service for six months. For warranty repair, send the product with copy of the original invoice to your dealer.

Replacement Turbine

for Midwest Quiet-Air Lever

Item Number 70903

High quality. Fully autoclavable. Made in USA.

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YI Europe, Ltd., Box 7037