# ATHENA CHAMPION®

## Replacement Cartridge Midwest Quiet Air or Insight Handpiece

*Item Number: 70950*

**IMPORTANT:** *Read these instructions before proceeding to replace the cartridge in a Midwest Quiet Air or Insight Handpiece.*

Before replacing the cartridge, carefully examine handpiece head for dents. If there are any dents in the head, return the entire handpiece to your dealer for repairs. Do not install replacement cartridge into a dented handpiece. Installation will permanently damage cartridge.

### REMOVAL

1. Unscrew the cap from the handpiece using the two pins on the “T” wrench, supplied with the replacement cartridge. Remove worn cartridge using the end of the “T” wrench.
2. Push worn O-ring and spring washer out of handpiece head using handle bar of “T” wrench. A dental explorer may also be used, if necessary.
3. Remove all foreign material and debris from interior of handpiece head. *Clean and lubricate interior of head with handpiece lubricant/cleaner.*

### INSTALLATION

1. Replacement cartridge should have an O-ring on the front bearing (located on the bur end). If O-ring is not on the bearing, install it with finger tips. Then, mount spring washer on top of O-ring. The winged edges of the spring washer should be pointing away from cartridge. If not already in place, insert cartridge into cap.
2. Holding cartridge (in cap) upright, screw replacement cartridge slowly into handpiece until slight resistance is encountered. Using “T” wrench, slowly turn cartridge in and out, while

*continued*

advancing 1/4 turn at a time, until replacement cartridge cap is flush against handpiece head.  
*Caution: Cap should be tightened securely with "T" wrench to insure safe positioning of cartridge.*

## BEFORE OPERATING HANDPIECE

1. Install bur securely into cartridge. Roll bur between fingers to make sure bearing turns freely. If any resistance, friction or grittiness is encountered installing replacement cartridge or bur does not turn freely between fingers...do not operate handpiece. Return entire handpiece and replacement cartridge to your dealer for installation.
2. Lubricate handpiece with handpiece lubricant/cleaner such as Athena Clave-N-Spray or Champ-Lube™ 20+ in accordance with daily maintenance instructions.
3. Connect handpiece to air/water supply and operate foot control. Do not operate without bur securely fastened in cartridge.
4. After handpiece has been put into service, lubricate, at least, once daily or in accordance with manufacturer's instructions.

*Warranty: This part is warrantied to the original purchaser against defective workmanship and materials under normal installation, use and service for six months. For warranty repair, send the product with copy of the original invoice to your dealer.*

INSTRUCTIONS FOR USE /  
MODO DE EMPLEO / MODE D'EMPLOI /  
GEBRAUCHSANWEISUNG / BRUKSANVISNING:  
[www.athenachampion.com](http://www.athenachampion.com)

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© 11/12 21100219A