

ATHENA CHAMPION™

2621 Corrinado Court • Fort Wayne, IN 46808
(800) 253-1771 • (260) 373-1917 • FAX (260) 373-0395

Athena Champion Push Button Canister

Highspeed Turbine

Item Number: 61902

- *Replacement Push Button Turbine*
- *Standard or Fiber Optic*
- *High-quality and Made in the USA*
- *Replaces Pana-air type cartridges*

INSTALLATION INSTRUCTIONS

Read these instructions before proceeding to replace the turbine.

Before replacing the turbine, carefully examine handpiece head for dents. If there are any dents in the head, return the entire handpiece to your dealer for repairs. **Do not install** replacement turbine into a dented handpiece. Installation will permanently damage turbine.

1. Remove cap by turning counterclockwise, using the cap wrench.
2. Remove worn turbine by pressing the blunt end of an inserted shipping post or bur against a hard surface to remove turbine.
3. Remove all foreign material and debris from interior of handpiece head. **Clean** and **lubricate** interior of head with 2 or 3 drops of Champ-Lube™ 20+.
4. Insert new turbine cartridge into handpiece head. When replacing with canister style cartridge, the guide projection on the canister must be lined up with concave groove inside handpiece head to ensure proper fit.

continued

5. Replace cap using finger to prevent cross threading. Once seated, tighten cap with cap wrench. Take care not to over tighten.

Sterilization Procedures

Remove the bur and any attachments from the handpiece. ***Do not use a sterilization bag.*** Put in steam or chemical vapor sterilizer and cycle. Autoclave/chemiclave up to 275° (135°C).

Lubricate Before Use

After sterilization and cool down time, insert bur and tighten with bur tool. Lubricate one more time with 1 or 2 drops of Champ-Lube 20+ and briefly operate to expel excess lubricant.

ACCESSORIES

Champ-Lube 2/Pack _____	30010
Champ-Lube 4/Pack _____	30015
Champ-Lube Plus Oiler Pen _____	30001
Replacement Push Button Cap _____	69015